

STYLE OPTIONS FOR THE BOSTON BAND®

Pattern or Color Transfer

- Scan for transfer options:

Bows

- Velcro or glue on

Stickers/Decals /Wraps

- BlingYourBand.com
- Etsy/Shop/OSoSplendid
- Signorama

Acrylic Paint

- Local Art Schools
- DIY



THINGS TO CONSIDER:

- Do not use a hot glue gun for glue
- Avoid spraying paint/airbrush on the the foam of the helmet
- It's recommended to seal the helmet with Mod Podge once styled